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Be aware of the the orthosis could influence i.e. the ability to drive a car. Be aware of that som pain can occur in muscles that has not been active when starting to use a new orthotic. Contact your orthotist or physical therapist.

ToeOFF® Fantasy
The Orthosis is offered in the following colours: Glossy Black, Glossy Dark Blue, Glossy Ivory, Glossy Purple. Contact your orthotist for availability in your area.

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GB

1. Do not "toe-walk" up stairs, or climb ladders on the ball of the foot.
2. Do not take too long steps.

3. Do not continuously squat to reach lower level.
4. Do not wear the device with footwear that has not been

approved by the orthotist.
5. Do not expose the device to high temperature extremes.
6. Do follow your orthotist's

instructions about use of the orthosis.
7. Do wear high cotton socks if your orthosis does not include other padding.

8. Do wash the liner/strap at least twice a week. Close Velcro into material prior to laundering.
9. Do check the condition of the foot, lower leg and the orthosis

on a daily basis.
10. Do report any abnormal condition of your skin, or changes in the orthosis, to the orthotist immediately.

ToeOFF® and BlueRocker™ should always have padding on the inside. SoftKit™ and SoftShell™ are two different kinds of padding options.

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© Camp Scandinavia AB / aug 05