

S.W.A.S.H.®

PELVIC SECTION

Ideal is to fit the orthosis at mid-pelvic girdle to obtain firm grasp and control of the pelvis. Depending on anatomy, final fitting location may vary from just below the waist (navel) to mid pelvic girdle.

THIGH CUFFS

Ideal is to fit the cuffs as distal as possible without creating pressure or interference in the popliteal area. However, pressure on the thorax, excessive spinal flexion, or discomfort due to hamstring tightness, may be relieved by moving the cuffs up 1-3 inches.

UPRIGHTS

Measure waist to midpatella. At final fitting, should be shortened to level of distal thigh cuff padding. 115 = narrow sitting base, 123 = wide sitting base.

NOTE: Color Coding indicates interchangeable components.

SIZING GUIDE

PELVIC SECTION

SZ	Sizes Available		Circumference Adjustment Range*	
	Classic	Low Profile II		
1	x	x	400 - 465mm	15 3/4 - 18 1/4in
2	x	x	460 - 550mm	18 - 21 3/4in
3	x	x	540 - 630mm	21 1/4 - 24 3/4in
4	x	x	620 - 720mm	24 1/2 - 28 1/4in
5	N/A	x	710 - 810mm	28 - 32in
6	N/A	x	800 - 900mm	31 1/2 - 35 1/2in

THIGH CUFFS

SZ	Sizes Available		Circumference Adjustment Range*	
	Classic	Low Profile II		
1	x	x	210 - 250mm	8 1/4 - 9 3/4 in
1A	x	N/A	210 - 250mm	8 1/4 - 9 3/4 in
2	x	x	250 - 290mm	9 3/4 - 11 1/2in
3	x	x	290 - 330mm	11 1/2 - 13in
4	x	x	330 - 380mm	13 - 15in
5	N/A	x	380 - 440mm	15 - 17in
6	N/A	x	440 - 500mm	17 1/4 - 19 1/2in

UPRIGHTS

SZ	Upright Diameter		Upright Length	
	Classic	Low Profile II		
1	6mm	8	310mm	12 1/4in
2	7mm	8	375mm	14 3/4in
3	7mm	8	440mm	17 1/4in
4	7mm	8	480mm	19in
5	N/A	8	540mm	21 1/4in
6	N/A	8	600mm	23 1/2in

PRODUCT NUMBER SELECTION

SWASH CLASSIC

SZ	PELVIC	CUFFS	UPRIGHTS	
			115 Deg.	123 Deg.
1	288000001	288030001	288010001	288020001
1A*	N/A	288030015	N/A	N/A
2	288000002	288030002	288010002	288020002
3	288000003	288030003	288010003	288020003
4	288000004	288030004	288010004	288020004


*Size 1 Cuff that fits Size 2 Upright

SWASH LOW PROFILE

SZ	PELVIC	CUFFS	UPRIGHTS	
			115 Deg.	123 Deg.
1	288350001	288420001	288400001	288410001
2	288350002	288420002	288400002	288410002
3	288350003	288420003	288400003	288410003
4	288350004	288420004	288400004	288410004
5	288350005	288420005	288400005	288410005
6	288350006	288420006	288400006	288410006

Classic

28801	Benstag 115°	C, H, O
28802	Benstag 123°	C, H, O
28803	Lårdel med textil	D
28804	Textil för midjedel	A
28805	Benstagsnav	K
28806	Plasthölje för benstagsnav	P
28807	Låsring för benstag	O
28808	Skruvset	E, F, G, L
28809	Mässingsbussning för lårdel	G
28810	Sölja med tryckknapp för midjedel	S
28811	Nit, skruv och bricka för midja	T
28812	Magband med tryckknapp	B
28813	Gummiplugg för benstag	H
28814	Textil lårdel	U
28815	Sölja med tryckknapp för lårdel	V
28820	Insexnyckel, set om 4	W

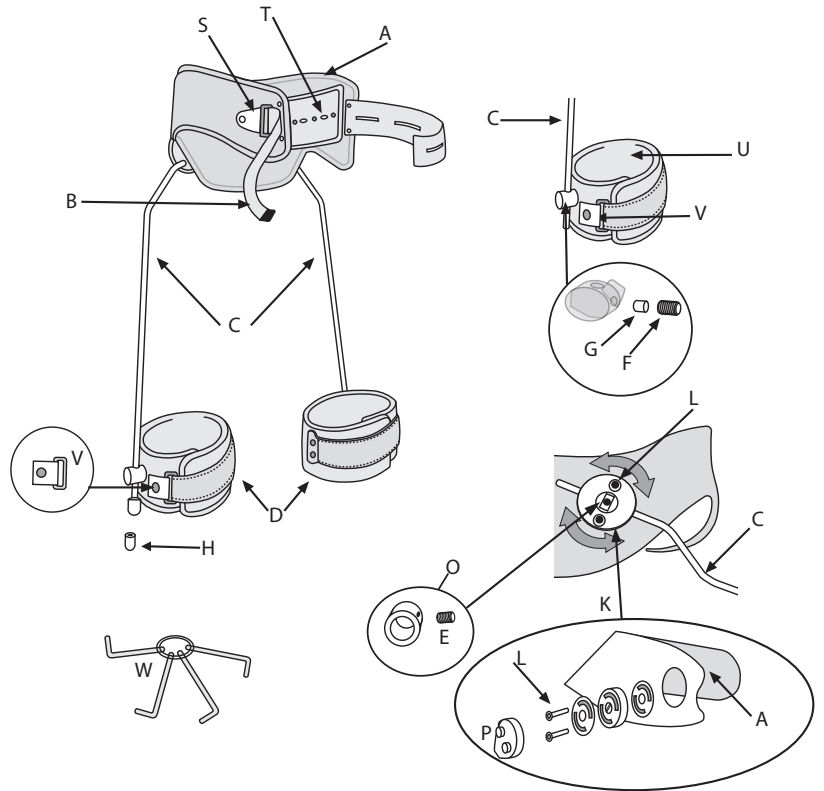

Low Profile II

28836	Textil midja inkl. magplatta	1, 2
28840	Benstag 115°	3, 8, 11
28841	Benstag 123°	3, 8, 11
28842	Lårdel med textil	10
28843	Textil lårdel	4
28844	Benstagsnav	6, 8
28845	Skruvset	7, 9, 12
28846/47	Magplatta	2
28848	Låsring för benstag	8
28849	Sölja med tryckknapp för lårdel	5
28813	Gummiplugg för benstag	11
28820	Insexnyckel, set om 4	W

